


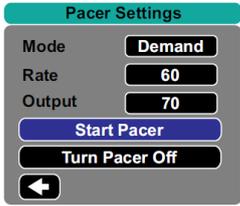

1	<p><b>Pacing is not currently authorised for any NAS Clinical Grade</b></p>	
2	<p>As the Pacing function is not disabled, it is possible that the function may be inadvertently switched on. This will cause the device to switch automatically to a Lead II view. If this unexpected event occurs during the management of a cardiac arrest it can effect normal defibrillation operations.</p>	
3	<p>If Pacing is activated, a pop-up “Pacer Settings” window will appear and the device will go into Pacing standing by mode, awaiting various settings to be adjusted and then started.</p>	
4	<p>If this happened in error, simply use the navigation keys to select “Turn Pacer off” this will revert to the last mode. If this pop-up Pacer setting menu has been cleared from the screen, press the Pacer button again to bring it back up and select “Turn Pacer off”.                  The Practitioner will have to manually select “Pads” from the lead select menu if managing a cardiac arrest.</p>	
5	<p>As an alternative method of exiting Pacer mode, press the Charge button, this will bring the device back into Defibrillator mode and exit pacing.</p>	