



Issue Date: 5<sup>th</sup> April 2018

## SAFETY ALERT NOTICE.

**Drug / Product / Device : Ferno Compact Chair.**

**Identified Issue / Concern / Problem/ Evidence of harm.**

On the 21<sup>st</sup> of February 2018 the seat in a ferno compact chair dropped as a bolt had become loose, this led to staff injury. Today 5<sup>th</sup> April a further chair seat fell apart after the same bolt loosened and fell out.

An identical nut and bolt on the opposite side of the seat were found to be so loose as to be just about to separate.

Both Chairs have a manufacture date of 2016 as indicated on the crossbar below the seat. It is not known at this point if this issue is confined to this year.


**Actions Required / Risk Treatment strategy** (include due date as appropriate).

Please inspect all Ferno Compact Chairs on all vehicles and including those which are in storage.

If any bolts are found to be loose or any other defects are noted, please arrange repair and withdraw from service in conjunction with supervisor and line manager. Please complete a defect form and a NIRF03 for any faults discovered.



Missing Bolt & Nut

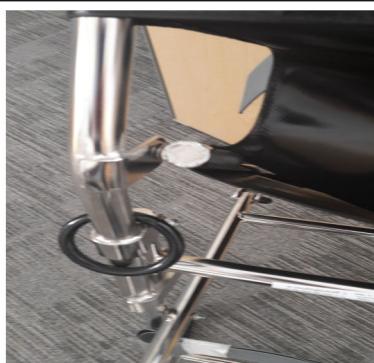

Bolt on opposite side loose



Nylock Nut which fell off

Issued by: Quality, Safety & Risk – North Leinster.